

# **SHEARTEX® ANKLE STIRRUP BRACE WITH MICROBAN®**

Model No: 40-901

Size: Universal  
Color: Navy



#### **INDICATIONS:**

- Ankle Instability
- Moderate Sprains
- Post Surgery
- Severe Strain

#### **BENEFITS:**

The Sheartex® Ankle Stirrup Brace provides medial and lateral support to stabilize the ankle joint. It also protects the medial and lateral ankle ligaments, while allowing full dorsi- and plantar-flexion. The rigid brace shells are designed to fit the ankle comfortably without any pressure or pinch points.

#### **PRODUCT FEATURES:**

- Sheartex® padding is extra soft for greater patient comfort
- Liner is completely air permeable to keep heat and perspiration to a minimum
- Liner can be easily removed for laundering
- Treated with Microban® - an additive that helps control the growth of certain odor and stain causing bacteria, yeast and fungi

#### **HOW TO APPLY:**

Place the rigid shells on the medial and lateral sides of the ankle. The heel width is easily adjusted and the brace is applied with two adjustable hook and loop straps. While squeezing side panels together, wrap the lower then the upper strap and fasten with hook 'n loop closures. Once both straps are fastened, re-adjust the lower then upper strap to ensure a proper fit. Fit should be snug, but not so tight it pinches or it constricts circulation. Wear over socks and with an athletic shoe.

#### **CARE INSTRUCTIONS:**

Sheartex® liners are easily removed and can be machine washed in cold water with mild soap. Do not bleach. Air dry away from heat. Do not tumble dry. Wipe shells with damp cloth.

#### **SIZING:**

This stirrup ankle brace has a universal design that fits most ankles, left or right.

800-327-4110