

GEL•FOAM™ ANKLE STIRRUP BRACE WITH MICROBAN®

Model No: 40-905

Size: Universal

Color: Navy



INDICATIONS:

- Ankle Instability
- Moderate Sprains
- Post Surgery
- Severe Strain

BENEFITS:

This stirrup ankle brace provides medial and lateral support to stabilize the ankle joint. This brace also improves ankle stability during athletic activity. The rigid brace shells are designed to fit the ankle comfortably without any pressure or pinch points.

PRODUCT FEATURES:

- Soft, terry-covered GelFoam™ liner
- GelFoam™ liner conforms to the ankle for intimate support
- GelFoam™ liner can be cooled in the refrigerator for use in cold therapy

HOW TO APPLY:

Place the rigid shells on the medial and lateral sides of the ankle. The heel width is easily adjusted and the brace is applied with two adjustable hook and loop straps. While squeezing side panels together, wrap the lower then the upper strap and fasten with hook 'n loop closures. Once both straps are fastened, re-adjust the lower then upper strap to ensure a proper fit. Fit should be snug, but not so tight it pinches or it constricts circulation. Wear over socks and with an athletic shoe. Liners can be removed and cooled in the refrigerator.

CARE INSTRUCTIONS:

Gel-foam™ liners are easily removed and can be hand washed in cold water with mild soap. Do not bleach. Air dry away from heat. Do not tumble dry.

SIZING:

This stirrup ankle brace has a universal design that fits most ankles, left or right.

800-327-4110