


# PROLITE® STIRRUP ANKLE BRACE WITH AIR LINERS

**Model No: 908**

**Sizes: Universal**  
**Color: Blue/White**

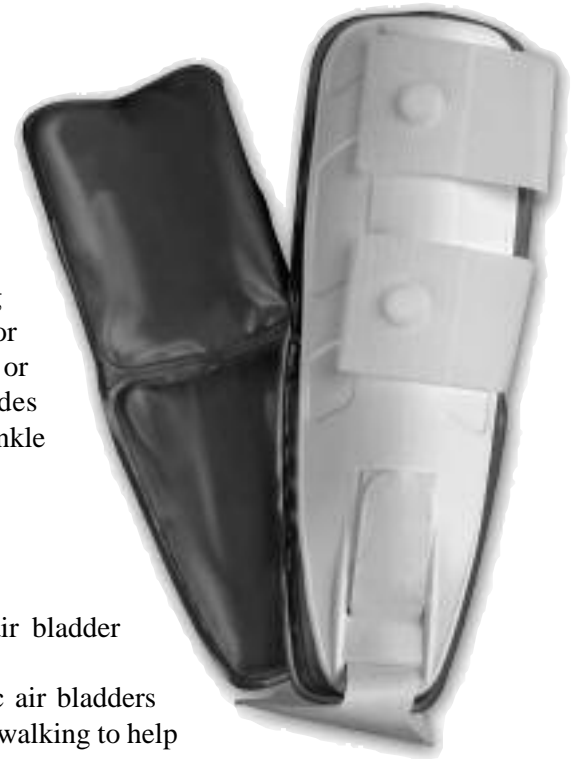

## USE:

This brace provides medial and lateral support to stabilize the ankle joint for faster recovery and healing of tender soft tissues. Ideal brace for mild or moderate sprains, strains, or after cast removal. Provides stabilization to chronically weak ankle joints as well.

## PRODUCT FEATURES:

- Unique, durable tri-layer foam air bladder construction liner
- Pre-inflated full length pneumatic air bladders pulsate and massage ankle during walking to help reduce ankle swelling and pain
- Stabilizes weak ankle joints following ankle sprains, strains, or after cast removal
- Adjustable heel piece conforms to fit a range of sizes
- Two adjustable swivel straps ensure proper strap placement for each application
- Standard height fits in most athletic shoes
- Fits most ankles, right or left.

## HOW TO APPLY:

To ensure support and function, stirrup braces should be worn with shoes, either with or without a sock. Heel width is corrected to give a snug fit by adjusting the heel stirrup strap. While squeezing side brace shells together against the leg, wrap the lower then the upper strap and fasten with hook and loop closures. Once both straps are fastened, re-adjust the lower then upper strap to ensure a proper fit. Fit should be snug, but not so tight it pinches or constricts circulation.

## CARE INSTRUCTIONS:

Spot clean with a soft cloth and mild soap. Rinse and wipe dry.



**800-327-4110**



Website: [www.flaorthopedics.com](http://www.flaorthopedics.com)